Steam Inhaler

with VapoPads

V1300EUV1

www.vickshumidifiers.co.uk

GB	STEAM INHALER WITH VAPOPADS	4
	USER INSTRUCTIONS	
DK	DAMPINHALATOR MED VAPOPADS	5
	BRUGERVEJLEDNING	
ES	INHALADOR DE VAPOR CON PASTILLAS VAPOPADS	6
	INSTRUCCIONES PARA EL USUARIO	
FI	HÖYRYSTIN VAPOPADS-TYNYNYLLÄ	7
	KÄYTTÖOHJEET	
FR	INHALATEUR À VAPEUR AVEC VAPOPADS...	8
	MODE D'EMPLOI	
IT	INALATORE A VAPORE CON PASTIGLIE VAPOPADS	9
	ISTRUZIONI D'USO	
NL	STOOMINHALATIEAPPARAAT MET VAPOPADS	10
	GEbruiksaanwijzing	
NO	DAMPINHALATOR MED VAPOPADS	11
	BRUKSANVISNING	
PL	INHALATOR PAROWY Z WKŁADAMI VAPOPADS	12
	INSTRUKCJA OBSŁUGI	
SA	جهاز استنشاق البخار مع شرائح VapoPads	13
	تعليمات المستخدم	
SE	ÅNGINHALATOR MED VAPOPADS	14
	BRUKSANVISNING	
TR	BUHAR SOLUYUCU İLE VAPOPADS	15
	KULLANICI TALİMATLARI	

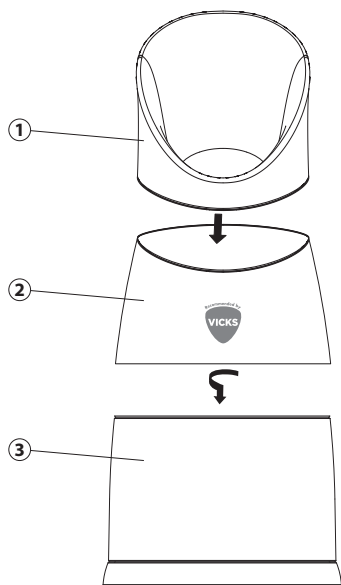
Fig. A

Fig. B

Fig. C

Congratulations on your purchase of the Steam Inhaler. Your new Inhaler provides all the benefits of vapour therapy at home, at the office or when traveling.

- Soothing menthol or rosemary & lavender vapours
- Portable – use anywhere
- Simply fill with hot tap water, add scent pad and breathe
- Dishwasher safe - reusable
- 2 VapoPads included

COMPONENTS (SEE PAGE 2)

1. Top 2. Middle Piece 3. Base

DIRECTIONS FOR USE

Rinse unit out thoroughly before first use.

1. Remove top of unit from base by twisting in opposite directions and lifting off top.


2. Fill base to water line with hot (not boiling) tap water. (See Fig. A on page 3)

NOTE: If you overfill, the unit will leak.

3. Open VapoPad packet and add to the water. Replace the top.

CAUTION: Product contains hot water. To avoid hot water spilling after filling, be sure to properly secure top housing by snapping back in place.

4. Breathe in the soothing vapours. (See Fig. B on page 3)

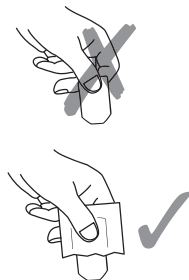CARE AND CLEANING

After each use, thoroughly wash and dry your inhaler. The unit is top shelf dishwasher safe. (See Fig. C on page 3)

USE WITH VAPOPADS

Your inhaler was designed to work with menthol or rosemary & lavender VapoPads.

WARNING:

Pad may present a choking hazard. Keep out of reach of children and pets.

H317 May cause an allergic skin reaction. P102 Keep out of reach of children. P302 + P352 IF ON SKIN: wash with plenty of soap and water. P333 + P313 If skin irritation or rash occurs: Get medical advice / attention

Ingredients: eucalyptus oil, menthol, glycol, cedar leaf oil & others

Do not use for babies less than 36 month old, for more than 16 hours in a 24 hour period and/or not more than 2 pads in a 24 hour period.

GUARANTEE

Your appliance is guaranteed against manufacturing defects for 2 years from date of purchase. Consumer Card available on our website at www.hot-europe.com/support. Please see last page of this manual to find the Support contact numbers in your country.

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© 2019, All rights reserved.