# Apicold

propo  
spray do nosa

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# Apicold

propo  
nasal spray

ISOTONIC SOLUTION FOR NASAL HUMIDIFICATION AND  
CLEANSING

## Medical device

**Composition:** AQUA, SODIUM CHLORIDE (Sea Salt),  
PROPOLIS EXTRACT, BENZALKONIUM CHLORIDE

## Intended use

Isotonic solution for nasal humidification and cleansing. Apicold® propo nasal spray normalizes the function of nasal mucous membrane on a natural way. Finely dispersed isotonic solution, allows action on the entire surface of the nasal mucous membrane. Its composition helps to establish the natural pH and nasal mucosa hydration, as well as the dissolution of dried mucous and accumulated dirt for easier breathing. The addition of propolis contributes to the stability of the product.

## It is recommended

- for daily hygiene of mucous membrane of children and adults
- while staying in air-conditioned rooms
- while exposed to dust and harmful atmospheric effects
- while staying in dry and warm rooms (central heating)
- for symptoms of allergic rhinitis and colds

## Instructions for use

Spray Apicold® propo nasal spray into both nostrils as required. One spray into each nostril is adequate to moisten dry mucosa. To remove dry secretion from the nose, spray Apicold® propo nasal spray into both nostrils liberally. Put absorbed cotton into the nostrils to prevent the liquid from flowing out until dry secretion softens.

## Caution

Keep out of reach of children. Not recommended for children under the age of 4. Due to hygienic reasons, it is recommended that one bottle is used for one person only. In case of hypersensitivity to certain ingredients, the use of Apicold® mint spray is not recommended. It is recommended to use the product within 30 days of opening. For external use only.

## Safekeeping

Store in dry and dark place at room temperature.

## Net quantity

30 ml

Lot number and expiry date are printed on the bottom of the packaging.

## Explanation of the symbols appearing on the packaging

	Consult instruction for use
	Expiry date
	Date of manufacture
	Warning
	Manufacturer
	Lot number
	Reference number
	Dispose carefully
	Product packaging is able to be recycled
	Flat cardboard

APIPHARMA d.o.o.  
Jeronima Kavanjina 26,  
10090 Zagreb,  
Croatia, EU

**Produced for:**  
Alpen Pharma AG,  
Casinoplatz 2, 3011  
Bern, Switzerland  
**Distributor:**  
Alpen Pharma  
Polska Sp. z o.o.,  
Al. Jerozolimskie 151/1,  
02-326 Warszawa

EN\_PL-10MD