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### Produktbeschreibung

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## EN Instructions for use

### Product description

Zetuvit Plus Silicone is a sterile absorbent dressing with a silicone interface. The dressing comprises a perforated silicone wound contact layer and a green absorbent pad. The silicone layer facilitates the application of the dressing and promotes non-adherence to the wound. Above the silicone layer, the product features a white, hydrophilic non-woven fabric (viscose and polypropylene). The absorbent pad comprises cellulose and superabsorbent polyacrylate (SAP) for the absorption and retention of exudate and is wrapped in a hydrophilic cellulose tissue to distribute the liquid. The backing of the dressing is made of green, polypropylene non-woven fabric to indicate the side facing away from the wound, which is hydrophobic but permeable to air and allows for gas exchange.

**Properties and mode of action**
Zetuvit Plus Silicone absorbs and retains up to high levels of exudate in the absorbent pad. The micro-abwärtige silicone interface prevents the dressing from slipping and facilitates its application. The dressing changes can be carried out atraumatically and almost painlessly.

**Intended purpose**
Zetuvit Plus Silicone is a sterile single-use absorbent dressing with a silicone interface for long-term treatment of acute and chronic exuding wounds. It is used on adults in clinical or homecare environments and can be combined with cleaning solutions and local antiseptics, primary dressings and additional fixation. It can be used in context of compression therapy. Zetuvit Plus Silicone can be used by healthcare professionals and lay persons under the supervision of healthcare professionals.

**Indications**
Zetuvit Plus Silicone is suitable for the treatment of a wide range of exuding wounds: acute wounds (burns, post-op, trauma wounds) and chronic wounds (ulcers, pressure ulcers, diabetic foot ulcers, tumour wounds).

**Mode of application**

- Cleanse the wound according to clinical practice. Make sure that the surrounding skin is dry and clean to ensure that the dressing sticks.
- Select the dressing's size and shape, so that the dressing exceeds the wound edges by approx. 1 to 2 cm.
- Do not cut the dressing.
- Remove the protective sheets according to the application instructions. Apply the adhesive side to the wound with the green side facing outwards.
- Secure the dressing using e.g. a Peha-half bandage or any other suitable fixation, or, if necessary, with compression bandages as instructed by a clinician.
- The dressing can remain on the wound for up to 7 days, depending on the condition of the wound and the clinician's decisions.
- The dressing must be changed if clinically indicated or when exudate approaches the edge of the dressing.

**Contraindications**
Do not use Zetuvit Plus Silicone in case of hypersensitivity to any of its components.

**Special precautions**

- Do not cut the dressing.
- Before treating a wound that is not healing as expected, consult a clinician. Treatment with Zetuvit Plus Silicone cannot replace a causal treatment of the wound healing impairment. If there are clinical signs of infection, the infection needs to be controlled with appropriate treatment before dressing can be used.
- In all cases, follow the clinician's instructions.
- In the absence of available data supporting the use of this dressing on sensitive population groups such as infants, children, pregnant or nursing women, and in the absence of data to the contrary, on these population groups this dressing should be used with caution and following a clinician's recommendation.
- It is not recommended to sand the dressing.
- Before opening the product packaging, inspect it carefully for signs of damage. Do not use the product if there are visible signs of damage, such as cracks, channels, pinholes, or nonuniform, torn, or incomplete seals.
- Reusing a single-use medical device is dangerous. Repressing devices, in order to reuse them, may seriously damage their integrity and their performance. Information available on request.

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UK – PAUL HARTMANN Asia-Pacific Ltd. - Kowloon, Hong Kong